

# Protocol Assist Device Update and Tutorial



**Established  
Status  
Epilepticus  
Treatment  
Trial**

Lindsey Harris, Research Technician Associate

# ESETT v1.3 Updates and Features

New features!

- Benzo check on Start Protocol screen
- PK/PD study reminder
- Fixed contact list





# Updating Protocol Assist Device iOS

1. Settings > General > Software Update
2. “Install Now” and agree to Terms & Conditions



# Updating ESETT App

1. Extras Folder > App Store > Updates > Purchased
2. Click “Update” box next to UVA ESETT app

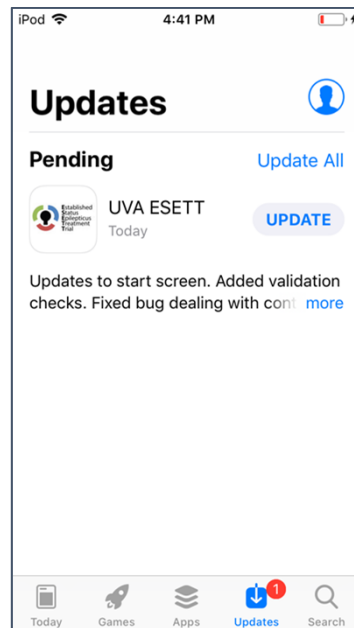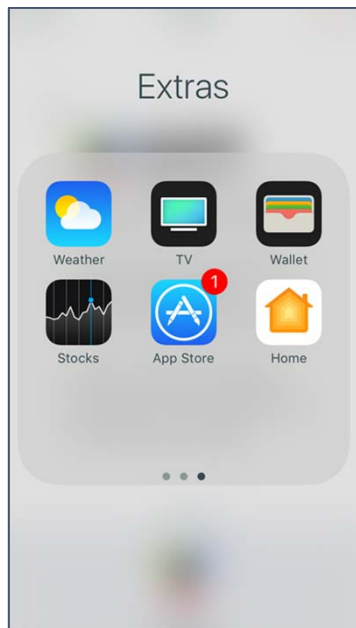


# Setting Up Study Drug Vial

Press ESETT icon




Vial needs scanned

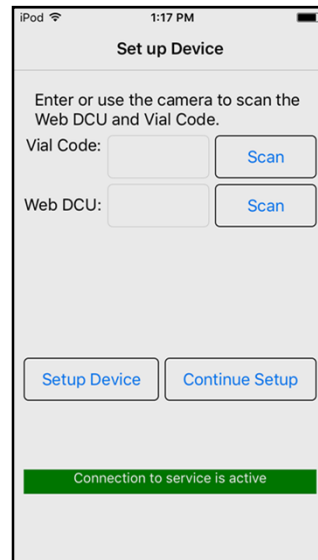


Or


Ready for enrollment

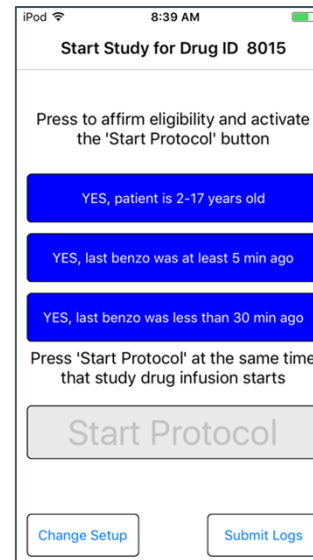

# Setting up Study Drug Vial

- Scanning barcodes
  - WebDCU: Drug Tracking > Drug Receiving > Vial number

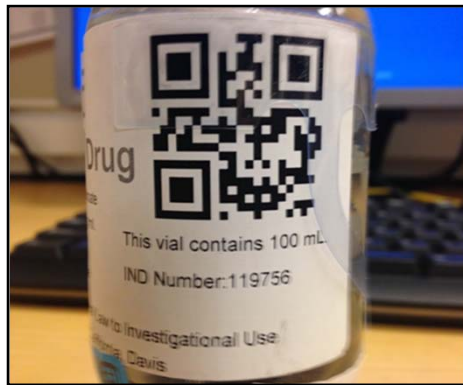






No.	Item Description	Data Value
ESETT study drug vials are color-coded with round stickers. The color of the sticker corresponds to the age group to which the vial is assigned, and matches the color of the "Use Next" box into which it should be placed. Purple - Younger than 18 Yellow - Between 18 and 65 years old Grey - Older than 65		
The vial with the black label is a backup vial which should be stored in your site's pharmacy. After a subject is enrolled and prior to receipt of a resupply vial, the backup vial should be temporarily placed in the used "Use Next" box. When the resupply vial is received, the backup vial should be returned to your site's pharmacy.		
1	Site	University of Michigan University Hospital, Ann Arbor, MI
2	Study Drug ID	9616
3	Color	Yellow
4	Age group	Adult (18-65)
5	QRC BarCode	 TEC.IT.COM
6	Received On	26-Jan-2016 (dd-mm-yyyy)
7	Receipt Notes	
8	Received By	Robert FINNEY

Last updated by Robert FINNEY on 26-Jan-2016 12:00AM

WebDCU™ © Copyright 2009-2016 Medical University of South Carolina. All rights reserved.





# Setting up Study Drug Vial

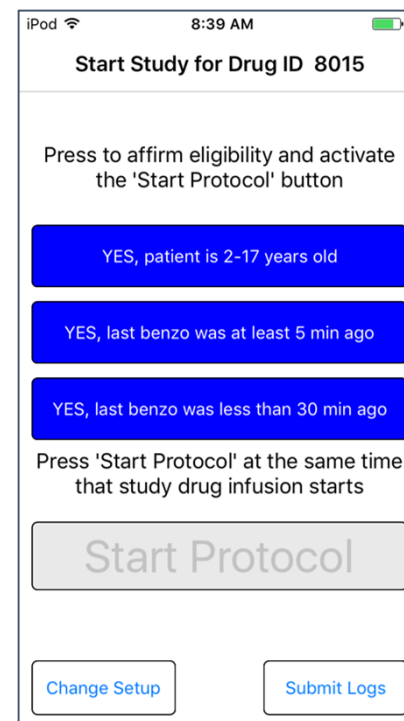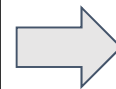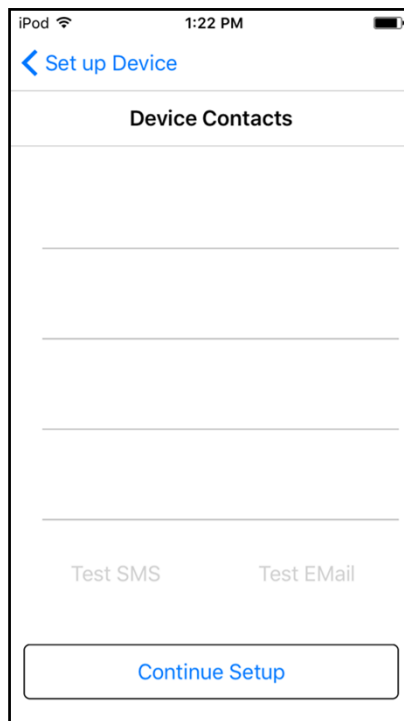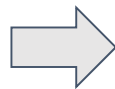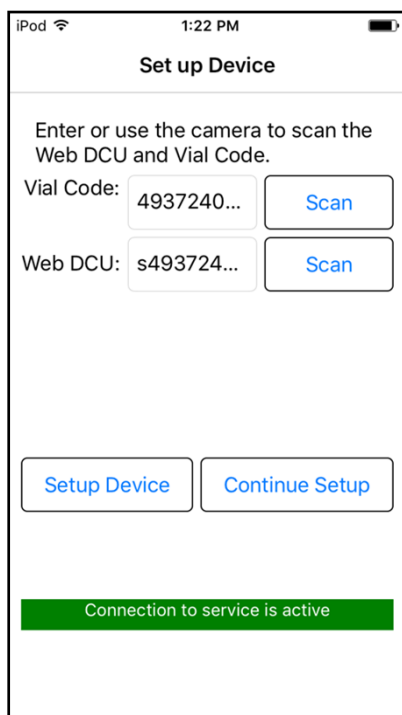



# Changing Vial Codes

- Press “Change Setup”
- Re-scan correct barcodes

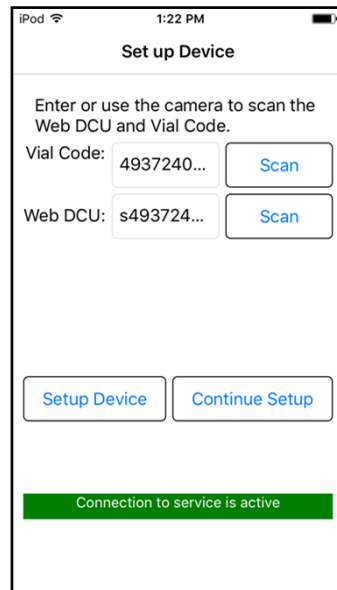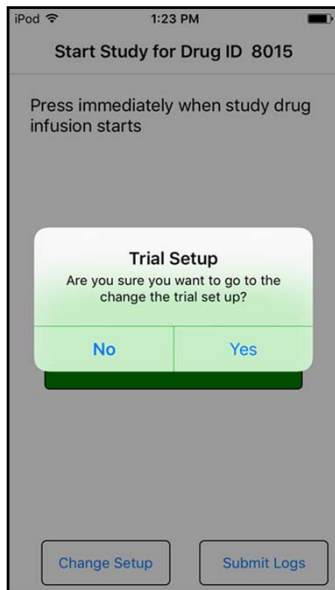






# PK/PD Acknowledgements

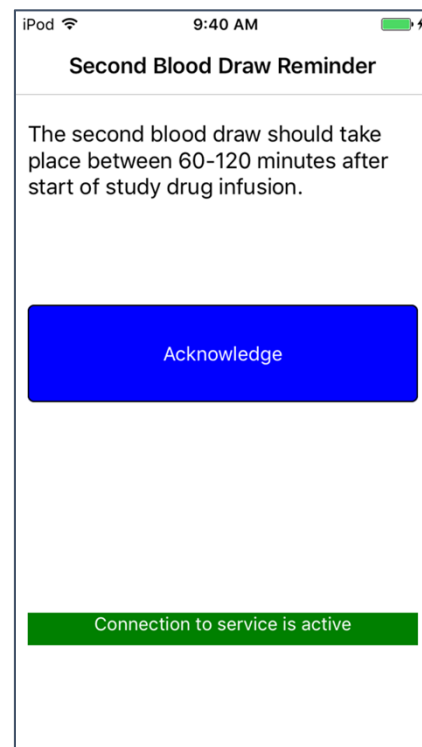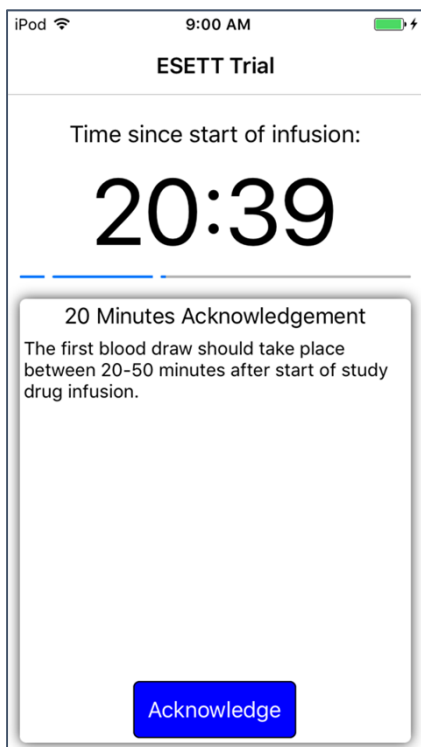




# Unblinding

**Reminder:** Unblind by scanning vial barcode

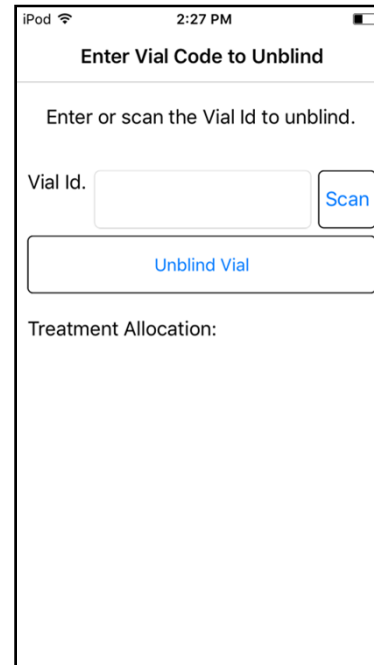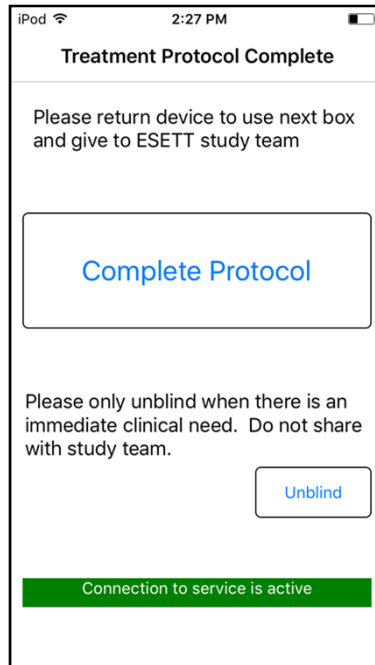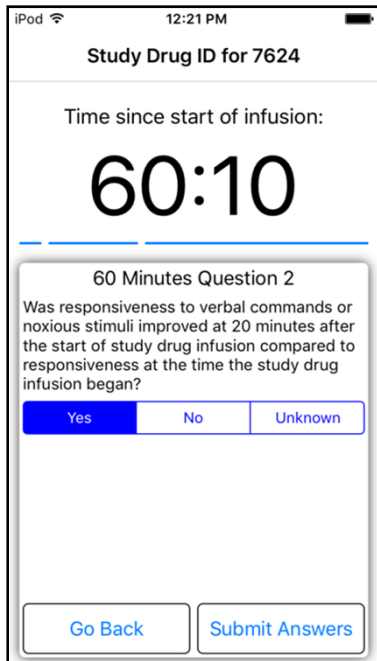

# Submitting Data

At this time, connect to Wi-Fi network if necessary.

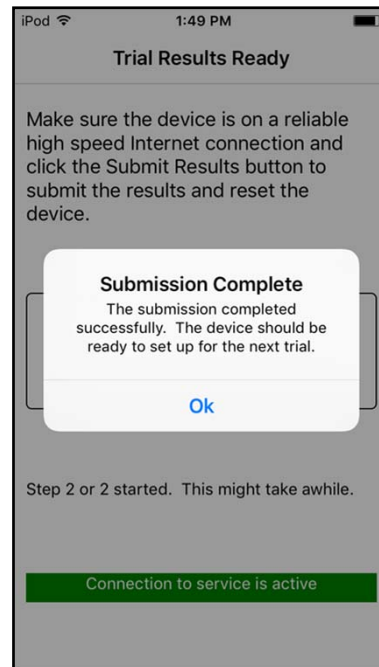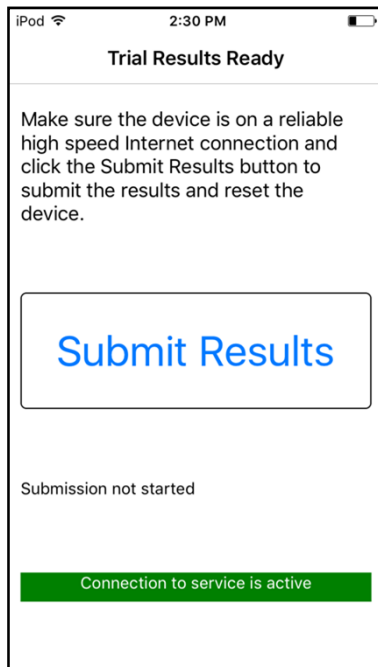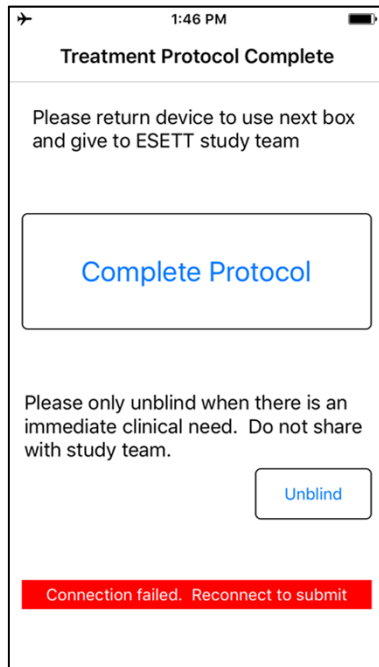

# Returning Use Next Boxes

- We'll contact you to return boxes
- Those who've already returned boxes - thanks!



# Resources

- Educational videos
  - ESETT > Study Team Only > Toolbox > Clinical Resources
- Protocol Assist Device User Manual
  - ESETT > Study Team Only > Manual of Procedures, sec 16.2

Or contact Lindsey Harris ([liha@umich.edu](mailto:liha@umich.edu)) or Nia Bozeman ([nbozeman@med.umich.edu](mailto:nbozeman@med.umich.edu)) for assistance.





**Established  
Status  
Epilepticus  
Treatment  
Trial**

[esett.org](http://esett.org)